

## QUICK INSTRUCTION



**With this unit you will operate directly on the patient. Therefore it is mandatory to go carefully through the manual in the box.**

Only skilled specialists are allowed to use the units. Patients must be informed about eventual risks prior to start the treatment.

### Symbols

---



**Warnings!**

---



**Upper temperature limit 50°C**

---



**Steam sterilization**

---



**Cartridge-content pressure 50 bar (caution: always use gloves while changing the cartridge)**  
**Cartridges are thermosensitive. Never & under no circumstances expose the cartridges to temperatures above 50° C.**

### Applications

Cryosurgery is suitable to treat almost all kind of skin lesions. E.g. warts, age pigments, hemangiolympangioma, fibroma, condyloma and more (see instruction manual).

### Attaching the cartridge

Put the metal pin in the intended hole in the unit body. This helps to grip the body. Screw the cartridge with the other hand clockwise quickly into the body. Take care to align straight and never force the thread. No noise of escaping gas should be heard whilst attaching the cartridge. Thereafter the unit is ready for use.

### Replacing the cartridge



**Always use gloves while changing the cartridge. You protect therewith your skin against an accidental gas contact.**

Use again the metal pin in order to unscrew empty cartridges in a counter clockwise direction. If there is any residual gas remaining, a relatively strong turn is necessary to unscrew. It does not make sense to unscrew cartridges still containing gas. Afterwards attach a new cartridge as described before.

### Working with the unit

Remove the protective cap and place the tip on the area to be treated. The liquid gas must be brought directly onto the skin, the contact between the tip and the skin is a MUST. The lever allows you to guide the gas flow. Instructions for the application times are found in the manual.

### Safety instructions



**Use the unit only as described in the instruction manual and only for the purpose as aforementioned.**

### Cleaning



Tips can be cleaned and disinfected with a disinfectant or alcohol. If they are in contact with blood, mucous or an infected area sterilization is a MUST prior to reuse them.

**Only the tips may be sterilized. ATTENTION!** Please follow the cleaning instructions in the manual. Blocked tips are not object of a warranty performance.



New Medical Technologies GmbH  
Freiburgstrasse 453 CH-3018 Bern

